



Hillrom™



Hillrom® 900 Accella™ bed

Enhance outcomes. Improve safety. Simplify tasks.



Hillrom™

## Hillrom 900 Accella bed

### FEATURES AND BENEFITS

#### Improve patient safety

- Reduce patient discomfort with **SlideGuard™**, especially designed to minimise the need for patient repositioning.
- Caregivers can respond more rapidly with **Auto CPR**, which flattens the back rest and automatically places the bed into a horizontal position, should patient complications occur.
- Alerts caregivers of potential bed exit via the **3-Mode Bed Exit Alarm**, available with remote monitoring capabilities\*

#### Enhance patient outcomes

- **One-Touch side egress** positions the bed for safer and easier bed exit.
- Stable support is provided during bed exit via an **ergonomically-placed side egress grip handle**.
- Optimise patient care with **Head-of-Bed angle alarm**, alerting caregivers of any safety concerns.
- Optional **Accella Therapy™ integrated surface** for enhanced protection against skin breakdown.

#### Simplify caregiver workflow

- **Simple weigh** improves workflow efficiency with an integrated scale and BMI calculation, excluding the weight of hanging accessories and IV poles, therefore reducing the steps needed to obtain patient weight.
- Bright and easy-to-use **Graphical Caregiver Interface (GCI)** offers straightforward visuals of patient information, such as head-of-bed angle and other caregiver alarms.
- **Intuitive bed controls** consistent with other Hillrom beds, requiring minimal in-service staff training helping to mitigate the risk of user error.



Auto CPR



3-Mode bed exit alarm



One-Touch side egress positioning



Accella Therapy integrated surface controls



Simple weigh function



Single connection in the bed frame easily connects to the Accella Therapy surface





HILLROM 900 ACCELLA BED TECHNICAL SPECIFICATIONS ANZ		
Safe working load		250 kg
Maximum patient weight (intensive and acute care)		185 kg
Height adjustment		43.1 cm to 80.8 cm
SlideGuard sliding backrest		up to 12 cm
Protection class		IPX4
Bed angles	Head section	0 to 65°
	Thigh section	0 to 28°
	Foot section	-3° to -22°
	Trendelenburg/reverse Trendelenburg	+17° to -17°
Sleep deck	Width	90 cm
	Length	200 cm
Siderail height		39.3 cm
Simple Weigh (Class III OIML)		●
Body Mass Index calculation		●
Bed extension with linen holder		●
Audible brake-off indicator		●
Intelligent night light		●
Accella Therapy connection ready solution		●
Controls	Integrated in siderails	●
	Pendant	○
	Bilateral high-low foot control	●
Graphical Caregiver Interface	Integrated in right siderail	●
Steering castor	At head end	●
Castors	150 mm double band	●
5th castor		○
3-Mode Bed Exit Alarm		●
Wired connection for 3-Mode Bed Exit Alarm		●
Wireless connectivity		○

● = standard ○ = optional

## ADDITIONAL STANDARD FEATURES FOR ALL CONFIGURATIONS

- 6 Attachment points for patient restraints
- Lifting/IV pole holders
- Unweighted bilateral drainage hanging bar with hooks
- Electric cord holder
- Adjustable mattress retainers
- Battery backup
- Bilateral mechanical CPR lever
- Removable head/footboards
- Head-of-Bed angle alarm
- EasyChair and flat position function
- One-Touch side egress
- Boost function
- High pressure laminate (HPL) sleep deck surface
- Central brake at foot end
- Low height indicator
- Electric CPR
- Bilateral mechanical & electric CPR
- Connectivity ready



**Hillrom™**



## **ABOUT HILLROM**

Hillrom is a global medical technology leader whose 10,000 employees have a single purpose: enhancing outcomes for patients and their caregivers by advancing connected care. Around the world, our innovations touch over 7 million patients each day. They help enable earlier diagnosis and treatment, optimise surgical efficiency and accelerate patient recovery while simplifying clinical communication and shifting care closer to home. We make these outcomes possible through connected smart beds, patient lifts, patient assessment and monitoring technologies, caregiver collaboration tools, respiratory care devices, advanced operating room equipment and more, delivering actionable, real-time insights at the point of care. Learn more at [hillrom.com.au](https://hillrom.com.au).

**To learn more, please contact your Hillrom Representative  
or Hillrom Customer Care on 1800 445 576.**

**Always read the labels and instructions for use.**

Please refer to HR900 User Manual 171051.

**AUTHORISED AUSTRALIAN SPONSOR Hill-Rom Pty Ltd.** Suite 4.01, 2-4 Lyonpark Road, Macquarie Park 2113 NSW Australia.

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale of its products.

© 2020 Hill-Rom Services, Inc. ALL RIGHTS RESERVED. APR222301-AU-r2 06-2021 ENG - AU

**[hillrom.com.au](https://hillrom.com.au)**

Not all features may be available for your country. Please contact your Hillrom representative for details.