



- Raise foot section
- Raise the head of bed
- Lower foot section
- Lower the head of bed
- Raise the HR900
- Side Egress
- Trendelenburg
- Lower the HR900 sleep deck
- Reverse Trendelenburg
- Easy Chair Position
- Press and hold UNLOCK KEY to activate Clinical Controls
- Returns the deck to flat position

UNDERSTANDING THE HAND CONTROL

Controls within the light blue section are identified as "Patient Comfort Controls". Controls within the dark blue section are "Clinical Controls". To activate the Clinical Controls, you **MUST** press and hold the **UNLOCK KEY** and then press the clinical control option you require.

SIMPLIFY TASKS AND IMPROVE PATIENT OUTCOMES

6 WAYS TO GET THE BEST FROM YOUR HR900 BED FRAME

1. When positioning a patient on the HR900, always ensure their hips align with the hip indicators located on the upper side rails. This ensures the HR900 bed frame Auto Contours for patient comfort and reduces patient migration down the bed.



2. Head of Bed angle indicators ensure the head of bed is at the correct angle, compliant with local facility policy. These can be found on either side of the top side rail.




3. In the case of an emergency, the CPR handles are located under the head section on both sides of the bed. To activate lift the yellow lever upwards until the head of bed is in the flat position.



4. Easy Chair – Simply press and hold UNLOCK and then continue to press the Easy Chair control. This allows you to position patient into a chair position, which has many clinical advantages.



5. One Touch Side Egress – Simply press and hold the Side Egress button.

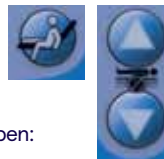


Three things happen:


- 1) Lowering of the foot section, allowing patient to rotate into a sitting position on the side of the bed frame.
- 2) Head of bed will lower or raise to place the top side rail in an ergonomic position for the patient to use the egress handle when exiting the bed.
- 3) The bed will lower to a suitable height to allow feet to be positioned correctly, for safe egress.



6. Intelligent Night Light – When the bed frame is positioned to the lowest height, the intelligent night light will change from orange to green. This can also be used to illuminate the floor for patients to see more clearly when leaving the bed in a darkened environment.

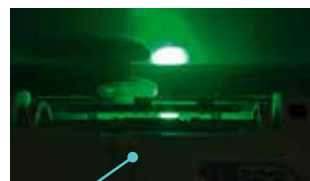

CLEANERS/DISINFECTANTS COMPATIBLE WITH BEDS AND SURFACES

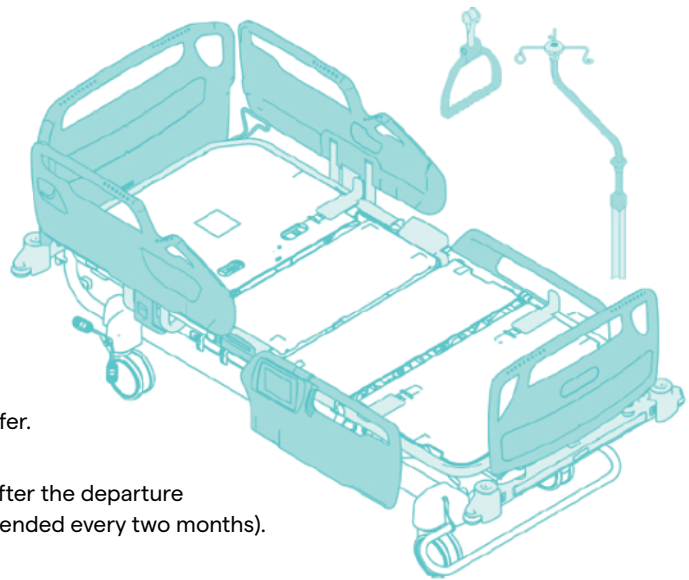

- Clean and disinfect every day.
- + Clean and disinfect after patient departure or before patient transfer.
- + + Clean and disinfect thoroughly (after the departure of an infected patient or recommended every two months).

List of Cleaners/Disinfectants for Routine Use

Cleaner/Disinfectant	Chemical Class	Recommended for Disinfection against Clostridium Difficile (C.Diff)*	Active Ingredient(s)
Oxycide™ Daily Disinfectant Cleaner	Hydrogen Peroxide	Yes	Hydrogen Peroxide 27.5% Peroxyacetic acid 5.8%
Virex® II 256 One-Step Disinfectant Cleaner and Deodorant	Quaternary ammonium chloride	No	Didecyl dimethyl ammonium chloride = 8.704% Alkyl dimethyl benzyl ammonium chloride = 8.19%
Virex® Tb Ready-to-Use Disinfectant Cleaner	Quaternary ammonium chloride	No	Diethylene glycol butyl ether 5-10% n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chloride 0.01 – 1.5% n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium Chloride 0.01 – 1.5%
3M™ HB Quat 25L Disinfectant Cleaner	Quaternary ammonium chloride	No	Alkyl dimethyl benzyl ammonium chloride = 13.238% Alkyl dimethyl ethylbenzyl ammonium chloride = 13.238%
Wex-Cide™	Phenolic	No	Ortho-Phenylphenol = 0.026% Ortho-Benzyl-para-Chlorophenol = 0.023%
Viraguard® Disinfectant Solution	Alcohol	No	Isopropyl alcohol = 70%
CSI Disinfectant Cleaner	Quaternary ammonium	No	n-Alkyl dimethyl benzyl ammonium chlorides = 0.105% n-Alkyl dimethyl ethylbenzyl ammonium chlorides = 0.105%

*Recommendation per Cleaner/Disinfectant Manufacturer's Instructions

List of Cleaners/Disinfectants for Occasional Use Only

Clorox Healthcare® Bleach Germicidal Cleaner	Bleach	Yes	Sodium hypochlorite 0.65% (6500 ppm) (1:10 dilution of bleach)
Clorox Healthcare® Bleach Germicidal Wipes	Bleach	Yes	Sodium hypochlorite 0.55% (5500 ppm) (1:10 dilution of bleach)

*Recommendation per Cleaner/Disinfectant Manufacturer's Instructions

Note: If bleach is used with another cleaner/disinfectant, use a new microfiber cloth soaked in tap water to remove any disinfectant residue prior to and after the bleach.

To learn more, please contact your Hillrom Representative or Hillrom Customer Care on 1800 445 576.



Always read the labels and instructions for use.
Please refer to HR900 User Manual 171051.

Hill-Rom Pty Ltd. Suite 4.01, 2-4 Lyonpark Road, Macquarie Park 2113 NSW Australia.

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale of its products.