


# Baxter

## Hillrom Progressa+ ICU BED

### FULLCHAIR POSITION

#### TO ACTIVATE:

- Make sure brakes are set.
- Press and hold the Chair control on the siderail or caregiver pendant. The bed transitions to the reclined position then to the Chair position.
- When the articulation stops and a tone sounds, the bed has reached the **FullChair** position.

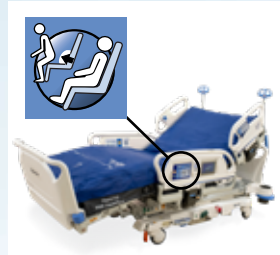


#### NOTE:

If the footboard is not installed, the bed will proceed into Chair Egress.

### CHAIR EGRESS

#### TO ACTIVATE:

- Make sure brakes are set.
- Remove the footboard if installed.
- Press and hold the Chair control until the bed reaches the **FullChair** position and lowers completely.
- For beds with an air surface, seat deflate will activate to lower the patient. Bed will beep three times indicating the surface is deflated.
- Press and hold the Chair Control again to proceed to the Chair Egress position.
- If needed, press and hold the Chair control to inflate the back section of the surface to assist with the patient's egress.

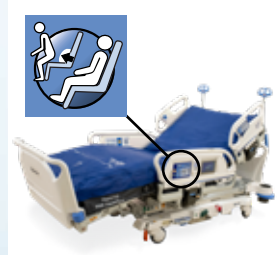

#### NOTE:

If the footboard is installed, when the bed reaches the **FullChair** position, the articulation stops and a tone sounds. Remove footboard to continue to Chair Egress.

### SIDE EXIT ASSIST

#### TO ACTIVATE:

- Raise or lower the bed so the patient's feet are flat on the floor.
- Press the Side Exit Assist control on the head end siderail.
- After the seat sections inflates, assist the patient with bed exit.



#### TO DEACTIVATE:

- Press the Side Exit Assist control on the head end siderail.

### BOOST POSITIONING SYSTEM

#### TO ACTIVATE:

- Press and hold the **Boost** control on the siderail.
- If the bed has an air surface, the surface will Max Inflate for 30 minutes and bed will transition to the Trendelenburg position.
- Release the **Boost** control when the desired position is reached.



#### NOTE:

To return to the flat position, press and hold the Bed Flat control. If the bed has an air surface, press the Normal control for the surface.

### NORMAL (CONTINUOUS LOW PRESSURE) MODE

Provides continuous full body pressure redistribution by automatically adjusting the surface to accommodate changes in weight distribution.

#### TO ACTIVATE:

- Select the surface tab on the Colour Touchscreen.
- Select Normal



### ALTERNATING PRESSURE MODE

Pressure redistribution mode provides pressure relief through cyclical change in loading and unloading in the head, seat, and foot zones of the surface.

#### TO ACTIVATE:

- Select the surface tab on the Colour Touchscreen.
- Select Alternating Low Pressure (ALP).
- ALP is active when the control turns green.
- To change cycle time, press the dropdown time menu and select the desired time (10, 15 or 20 minutes).




#### TO DEACTIVATE:

- Select the surface tab on the Colour Touchscreen.
- Press Normal.

### MAX INFLATE


#### TO ACTIVATE:

- Max Inflate times out after 30 mins
- Select the surface tab on the Colour Touchscreen.
- Select P-Max Inflate.
- Alternatively, press the P-Max Inflate control on the siderail.

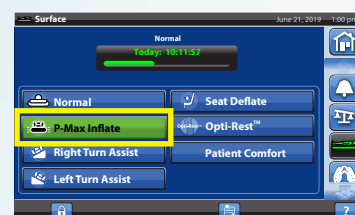

#### TO DEACTIVATE:

- Select the surface tab on the Colour Touchscreen.
- Press Normal.
- Alternatively, press the P-Max Inflate control on the siderail.

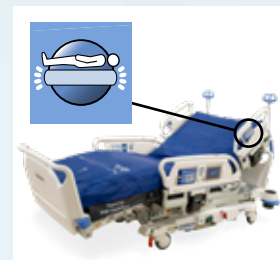

### TURN ASSIST

#### TO ACTIVATE:

- Select the surface tab on the Colour Touchscreen.
- Press Right or Left Turn Assist; the control turns green when active.
- To stop Turn Assist, press Normal.
- To pause the turn at less than the full angle, press the Stop Turn control while the Turn Assist is inflating.



#### TO DEACTIVATE:

- Select the Surface tab on the Colour Touchscreen.
- Press Normal.

### CONTINUOUS LATERAL ROTATION THERAPY

#### TO ACTIVATE:

- Select the Pulmonary Therapy tab on the Colour Touchscreen.
- Press Rotation.
- Select Full, Moderate, or Minimum. For custom options, select Custom (sets rotation direction, rotation percentage, and duration).
- If applicable, press Preview to check the patient's IV lines and tubes prior to starting rotation therapy.
- Press Start to begin Rotation.



#### TO DEACTIVATE:

- Select the Pulmonary Therapy Tab on the Colour Touchscreen.
- Press Rotation.
- Press Stop Therapies.



### PERCUSSION AND VIBRATION (P&V)


#### TO ACTIVATE:

- Select the Pulmonary Therapy tab on the Colour Touchscreen.
- Press P&V.
- Select High, Medium, or Low. For custom settings, select Custom (sets intensity, frequency and duration).
- Press Start to begin P&V.



#### TO DEACTIVATE:

- Press the Pulmonary Therapy tab on the Colour Touchscreen.
- Select P&V.
- Press Stop Therapies.

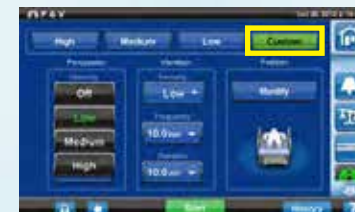

### BED EXIT ALERT

#### TO ACTIVATE:

- Ensure the patient is centered on the bed and aligned with the hip indicator.
- Select the Alerts tab on the Colour Touchscreen.
- Press Bed Exit.
- Select Position, Exiting, or Out of Bed mode:
  - Position – alerts when patient changes position
  - Exiting – alerts when patient moves away from the centre of bed toward edge of bed
  - Out of Bed – alerts when patients weight has left the bed
- When the bed exit system is activated, and it detects and alert condition, an alert will sound and a message will appear on the colour touchscreen.





#### TO DEACTIVATE:

- Press the Alerts tab on the Colour Touchscreen.
- Press Bed Exit.
- Press Off.

#### SILENCE BED EXIT SYSTEM WITHOUT DEACTIVATING SYSTEM:

- To Activate Preemptive Alert Silence
  - Press the Silence control located at the lower left of the colour touchscreen. This will allow patient movement or procedures to be done without the alert sounding.
- To Silence a Bed Exit Alert
  - When the bed exit system is activated and it detects an alert condition, an alert will sound and a message will appear on the color touchscreen.
  - Press Silence. Next, select Resume, Suspend, or Alert Off:
    - Resume – immediately turns on the bed exit alert
    - Suspend – allows 10, 20, or 30 minutes more time before the bed attempts to rearm the alert
    - Alert off – turns the bed exit alert off

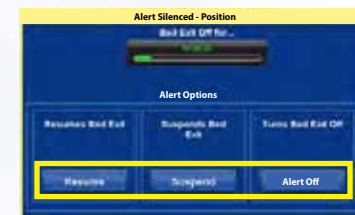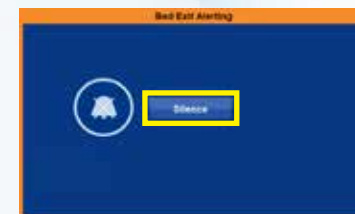

# TIPS & TRICKS

### SCALE

#### ZERO THE SCALE:

- Make sure the patient is not in the bed.
- Select the Scale tab on the Colour Touchscreen.
- Press Zero and follow the on-screen instructions.
  - New Patient
    - Erases Scale history
    - Zeros the scale
    - Reset all settings
  - Zero
    - Does not erase Scale History
    - Zeros the scale




#### TO WEIGH THE PATIENT:

- Ensure the patient is in the centre of the bed.
- Select the Scale tab on the home screen.
- Press Weigh Patient then follow on-screen instructions.
- If the bed is not in the recommended position, the message "Not Required Position" message will show on the Colour Touchscreen; adjust the bed position as applicable.



#### TO VIEW WEIGHT HISTORY:

- Select the Scale tab.
- Press History tab.

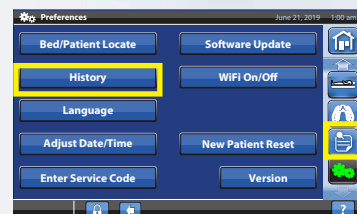

### PATIENT HISTORY

- Press the Settings menu tab on the Colour Touchscreen.
- Press History.
- Select the desired history to be viewed:
  - Bed Exit – Displays the time spent with the Bed Exit alert on
  - Head Angle – Displays time spent with the head of bed more than 30 or 45 degrees since 12am with Head Angle alert active
  - Scale - Displays the saved weights in the 24-hour period
  - Rotation - Displays the maximum number of cycles/hour the patient has rotated and hrs: mins in rotation in 24 hour period
  - P&V - Displays the number of Percussion and Vibration treatments provided per 24-hour period
  - Chair – Displays time Spent in Chair position since 12am
  - Opti-Rest - Displays time spent in Opti-Rest mode since 12am

FOR COMPLETE INSTRUCTIONS ON HOW TO OPERATE THE HILLROM PROGRESSA+ BED SYSTEM, PLEASE REFER TO USER'S MANUAL.

For more information, please contact your **Baxter** representative or call **Baxter Customer Care** on **1800 445 576**.

**ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE** CONTACT YOUR BAXTER REPRESENTATIVE FOR THE LATEST VERSIONS

Baxter, Boost, FullChair, Hillrom, and Progressa+ are registered trademarks of Baxter International, Inc, or its subsidiaries.

AUTHORISED AUSTRALIAN SPONSOR Hill-Rom Pty. Ltd. 1 Baxter Drive, Old Toongabbie NSW 2146.

hillrom.com.au

AU-CS390-240002\_r1 02/2024