

# Centrella™ Smart+ Bed

## Model CNT255 Skills Checklist

Staff Signature: \_\_\_\_\_

Trainer's Signature: \_\_\_\_\_

Date: \_\_\_\_\_



Perform the following.

COMPLETED

### Frame Features

1. State the maximum <b>patient weight limit</b> .	
2. Unplug bed's <b>power cord</b> . Identify the one (1) audible and two (2) visual cues when the bed is unplugged from power.	
3. Demonstrate use of <b>battery backup</b> to raise head. Identify controls available on battery backup. Identify % battery 'charged' indicator for bed frame.	
4. Lock Head control and unlock <b>retractable foot</b> feature and shorten the bed to shortest position. State all controls that can be locked out.	
5. Explain how bed indicates the <b>communication cable/call light</b> is not connected. There are two (2) audible and two (2) visual indicators.	
6. Lower the head and intermediate <b>siderails</b> on one side of the bed. Raise into a locked position.	
7. Identify the one (1) audible and one (1) visual cue that the bed is in the <b>lowest position</b> .	
8. Elevate the <b>lower legs/foot-of-bed</b> to be parallel to the floor. Explain when this may be useful. Slightly lower feet and then level the legs.	
9. Using <b>CPR control</b> , position bed for CPR. Identify situations where a CPR board may be recommended.	
10. Remove <b>headboard</b> for emergency access to the head of the bed.	
11. Identify two (2) <b>head-of-bed indicators</b> on the bed.	
12. Identify the built-in <b>siderail line managers</b> .	
13. Identify the <b>hip indicator</b> label for patient placement in the bed. Explain why this is important.	
14. Point to the three (3) areas for attaching <b>restraints</b> on one side of the bed.	
15. Identify the <b>foley/drainage bag holder</b> on one side of the bed.	
16. Explain the <b>foley position limit</b> . Demonstrate how to turn it on and how to recognize it is active.	
17. Explain the difference between when the bed sounds a <b>single beep</b> and a <b>triple beep</b> .	
18. Locate the <b>Boost® control</b> . Explain when it is used and what happens when Boost is activated.	
19. Explain what the three (3) lighted <b>SafeView®+ (optional) icons</b> (at the foot-end of the bed and on the floor) indicate. Why are there blinking lights? Correct the situation to stop the lights from blinking.	
20. Explain purpose of <b>lighted siderail grip (optional)</b> . What turns it on? Explain meaning when light is blue? Amber? Flashing Amber?	
21. Locate and explain purpose of the <b>night light</b> . Identify what activates the brightening of the night light.	
22. Locate these <b>patient accessible siderail features</b> : Siderail storage for devices, urinal holder, and head-of-bed angle indicator.	

# Centrella™ Smart+ Bed

## Model CNT255 Skills Checklist

### Perform the following.

COMPLETED

23. Explain the <b>patient remote controls</b> : nurse call, bed exit 'on' reminder, stand assist. Hang patient remote on all three (3) siderail locations.	
24. Demonstrate putting <b>brake</b> pedal into <b>steer</b> mode and then into <b>neutral</b> .	
25. Demonstrate using the <b>IntelliDrive® Powered Transport (optional) system</b> by moving the bed forward and backward. Demonstrate how to move bed side-to-side, as if to straighten in a room or up against a stretcher. State % of battery charge.	

### GCI® Touchscreen

26. Define the three-levels of <b>bed exit</b> sensitivity.	
27. Set the <b>bed exit</b> sensitivity to alert before a patient gets out of the bed. State the four (4) ways you know bed exit is ON (patient in bed).	
28. Pre-emptively <b>SILENCE</b> the <b>bed exit alert</b> . Explain how to add time for Silence period and the meaning of <b>triple beep</b> .	
29. Resume the <b>bed exit alert</b> . Identify two (2) ways to do this.	
30. Explain the difference between the <b>scale</b> 'new patient zero' and 'same patient re-zero.'	
31. <b>Zero scale</b> for 'new patient.' Identify three (3) screens you can check to see if the scale was zeroed.	
32. <b>Weigh patient</b> . Demonstrate how to display weight in lbs, then save weight in kgs. Recall weight after saving.	
33. Set the <30° <b>head-of-bed limit</b> . Turn off function.	
34. Locate the <b>stand assist</b> control on the GCI® Touchscreen.	
35. Demonstrate putting the bed into the <b>chair position</b> . Explain meaning of <b>triple beep</b> .	
36. Level the bed from the chair position using the <b>bed flat</b> control.	
37. Locate the <b>settings</b> screen. Turn OFF the SafeView® alert lights.	
38. Turn off the <b>voice alert</b> . Identify voice alert that cannot be turned off.	
39. Locate the <b>help screens</b> and <b>service wrench</b> on the GCI® Touchscreen.	

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2017 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.  
203848 rev 1 03-AUG-2017 ENG – US