



Skills Checklist

Staff Name (PRINT): _____

Trainer's Signature: _____

PERFORM THE FOLLOWING.

COMPLETED

INITIALS

	COMPLETED	INITIALS
1. Demonstrate and explain the available bed features to aid in transferring patients.		
2. State the weight capacity of the lift, and demonstrate use of hand controls and safety features — including emergency operations and storage recommendations.		
3. Explain the hospital policy for assessing and determining patient dependency and process for communicating transfer needs (equipment and slings).		
4. Demonstrate seated sling application.		
5. Demonstrate how to perform a pre-lift safety check of the sling straps. Straps should be secured properly in the sling bar hooks. Safety latches should be in place. Sling straps should not be twisted.		
6. Demonstrate the proper lifting height by raising the patient just enough to clear the bed or chair surface.		
7. Demonstrate seated transfer from the bed to the chair/toilet/stretchers.		
8. Demonstrate seated sling removal with minimal manual lifting or skin friction.		
9. Demonstrate turning the patient from side to side and elevating in bed using the repositioning sheet.*		
10. Demonstrate supine lateral transfer to the stretcher.*		
11. Demonstrate ambulation with LiftPants or MasterVest .*		
12. Demonstrate transfer from the floor.		
13. Demonstrate the use of the MultiStrap for limb holding, turning in bed, and foley catheter insertion.*		
14. Demonstrate the use of MultiStrap for proning.		

*if applicable

PERFORM THE FOLLOWING.

COMPLETED INITIALS

	COMPLETED	INITIALS
15 Demonstrate automobile transfer from both the front and backseat (emergency department staff).*		
16 Explain why it is essential to never leave the patient unattended during a transfer.		
17 Explain the importance of plugging in the lift when it is not in use.		

*if applicable

I have read and understand the associated Lift and Sling Instruction Guidelines.

Signature: _____

Date: _____

For safe and proper use of the product mentioned herein, please refer to the appropriate Operator's Manual or Instructions for Use.

Baxter reserves the right to make changes without notice in design, specifications and models. The only warranty Baxter makes is the express written warranty extended on the sale or rental of its products.

bax.to/sphm

Baxter, Golvo, LiftPants, MasterVest and MultiStrap are trademarks of Baxter International Inc. or its subsidiaries.

US-CS123-250008 (v1.0) 05/2025